

A comparative study of radiation dose and screening time between mini C-arm and standard fluoroscopy in elective foot and ankle surgery

Edward J.C. Dawe, Ernest Fawzy, Jakub Kaczynski, Phillip Hassman, Simon H. Palmer
Foot and Ankle Surgery 17 (2011) 33–36

Objective

This study compares the radiation dose delivered and screening times used between standard fluoroscopy and the mini C-arm during foot and ankle surgery. It also provides an estimate of potential cost savings in GBP as of 2009.

Method

- 127 cases who required intra-operative screening during various elective foot and ankle procedures were prospectively reviewed. Mini C-arm (Hologic, InSight 2) was used in 55 patients and was surgeon operated. Standard fluoroscopy (Siremobil 2000, Siemens Medical Systems) in 72 patients and was radiographer operated.
- Dose Area Product (DAP) was used as a measure of dose.
- Costs of standard fluoroscopy was determined by calculating the cost of providing a radiographer to theatre as well as the cost of delays to theatre caused by radiographer in attendance. Radiographer cost as £30/h. Mini C-arm cost was £42,500. Cost of theater time £15/min. Surgeon mini c-arm training cost was £350 each.

Results

There was a statistically significant reduction in mean DAP using the mini C-arm, 3.46 Gy cm² vs 7.43 Gy cm² (P = 0.0013). Table below shows the procedures and the DAP values for standard and mini c-arm fluoroscopy. There was no statistical difference in screening time.

	Standard Fluoroscopy		Mini C-arm	
	Frequency	DAP	Frequency	DAP
Forefoot procedures	21	2.9	19	2.36
Steroid injection	16	7.56	17	3.87
Examination of anesthesia of ankle	15	4.75	7	2.77
Hindfoot procedures	14	26.9	10	4.40
Subtalar arthrodesis	5	6.60	2	8.14
All procedures	71	5.90	55	3.32

The saving that could potentially be associated in 2009 with not using radiographers for extremity procedures is £5541 in radiographer delays and £3840 in radiographer salaries. The annual savings in 2009 from using the mini C-arm could be £9391, saving the total cost of the device over 5 years.

Conclusion

- Mini C-arm use in elective foot and ankle surgery gave a significant reduction in radiation use when compared to standard fluoroscopy. No statistically significant difference was observed between the screening times for the two groups. The introduction of a mini C-arm potentially reduces the cost and its use is recommended in foot and ankle surgery.